

Installation

Place and centralize the platform directly onto the bed frame and then place the mattress or mattress system directly on top.

⚠ IMPORTANT The platform should be free to inflate. Please ensure that each side of the platform can freely inflate and not be restricted by the mattress or the bed frame.

Toto Touch Control Unit

Using the bed hooks located on the rear of the control unit, securely hang on the footboard of the bed. Alternatively, position on any stable horizontal surface. Connect to mains power via the power lead supplied.

⚠ IMPORTANT A Healthcare Professional should determine inflation and rest durations.

Connecting the Platform to the Control Unit

Using the quick connectors, connect the grey inflation hoses to the platform air cell hoses – red to red.

Connect the grey inflation hoses to the control unit.

Check the CPR deflation valves are securely in place.







Operating Instructions

- To turn the unit on, press and hold the power button (1) for three seconds.
The default settings are:
Air cell side selection (2) – both sides
Air cell inflation time (10) – 120 minutes
Air cell rest time (9) – 120 minutes
- To change the default settings, press the button to scroll through the LED lit options until you have arrived at your preferred choice.
- Air cell inflation begins, and each side remains inflated for the duration that is selected.

Note - If the control unit is 'turned off' it reverts to the default settings when switched back on. Reselect the preferred options using the above instructions.

Cleaning and Care

All components can be cleaned and decontaminated with either a 0.1% or 1% chlorine-based solution when followed by a thorough rinse with clean water. Please ensure that the control unit is switched off and unplugged before cleaning begins.

Problem	Cause	Initial Checks / Trouble Shooting
 No LEDs lit; no audible alarm.	<ul style="list-style-type: none"> Control unit may not be attached to a power source. Fuse may need replacing. 	<ul style="list-style-type: none"> Check the control unit is connected to an active mains power outlet which is switched on. Check the control unit is switched on. If following points 1 and 2 does not resolve the problem, switch off and unplug the control unit: <ul style="list-style-type: none"> - Check the mains plug fuse (3A). - Check the control unit fuse (1A). Restart the control unit.
 Air leak (7) permanently lit yellow; an audible alarm can be heard.	<ul style="list-style-type: none"> Pressure has fallen below the minimum operational requirements. 	<ul style="list-style-type: none"> Press the alarm mute button. Check the CPR valve is properly closed, ensuring all sealing connectors are fully pressed in. Check each air cell is securely attached to the connecting air cell hose. Check all air cells and hoses for any air leakage.
 Air blockage alarm (5) permanently lit yellow; an audible alarm can be heard.	<ul style="list-style-type: none"> Air flow between the control unit and the platform is obstructed. 	<ul style="list-style-type: none"> Press the alarm mute button. Check for twists or kinks in the air hoses between the platform and control unit. Resolve and wait for the alarm to reset.
 Service LED (6) permanently lit amber.	<ul style="list-style-type: none"> Service required. 	Please contact Frontier Therapeutics for support Tel. (800) 303-9544. The Control Unit will continue to function even though the service LED remains lit.
 Interface panel is lit but unresponsive.	<ul style="list-style-type: none"> Interface panel is locked. 	<ul style="list-style-type: none"> Check that screen lock LED is lit amber (4). Press and hold the alarm mute button for four seconds.
 Audible alarm sounding; no LEDs lit.	<ul style="list-style-type: none"> Accidental removal of power lead. Accidental mains wall power is turned off. Mains power failure. 	<ul style="list-style-type: none"> Reconnect mains power lead to the control unit. Check the mains power wall switch is turned on. Attempt to restart the control unit.



Troubleshooting Not Working?

If you require any further assistance with the troubleshooting or repair of your Toto System please call the number below.

Additional information can be found on our website www.frontier-group.co.uk



Frontier Therapeutics Limited

300 International Drive, Suite 100 Buffalo,
NY 14221 United States of America

(800) 303-9544
contactus@frontier-therapeutics.com
www.frontier-therapeutics.com



